

MonoMouse-USB-MD™

Bierley

1 Year Worldwide Warranty

The Bierley MonoMouse-USB-MD carries a one year worldwide warranty, effective from the original date of purchase by the consumer.

For warranty claims, the complete product and accessories must be returned with the original purchase receipt to your local supplier or directly to Bierley in the USA.

Warranty void in the event of damage or abuse caused, in the opinion of Bierley, by general wear and tear, abuse, impact, neglect.

Bierley Associates, Inc.

19500 Graystone Lane
San Jose, CA 95120
USA

Tel 1 800 985-0535
E-mail ussales@bierley.com
Web www.bierley.com

MonoMouse-USB-MD™



www.bierley.com


www.bierley.com

Installation : First - Software

- 1) Unplug any existing camera devices and insert the CD in your computer. XP, Vista or Windows 7.
- 2) Launch "Bierley MonoMouse-USB-MD.exe"
- 3) Click "Next" as necessary and then "Finish"

Second - Hardware

Simply plug your MonoMouse-USB-MD into any USB 2.0 port on your computer**. The hardware will install automatically and a notification will appear confirming you new device is ready to use.

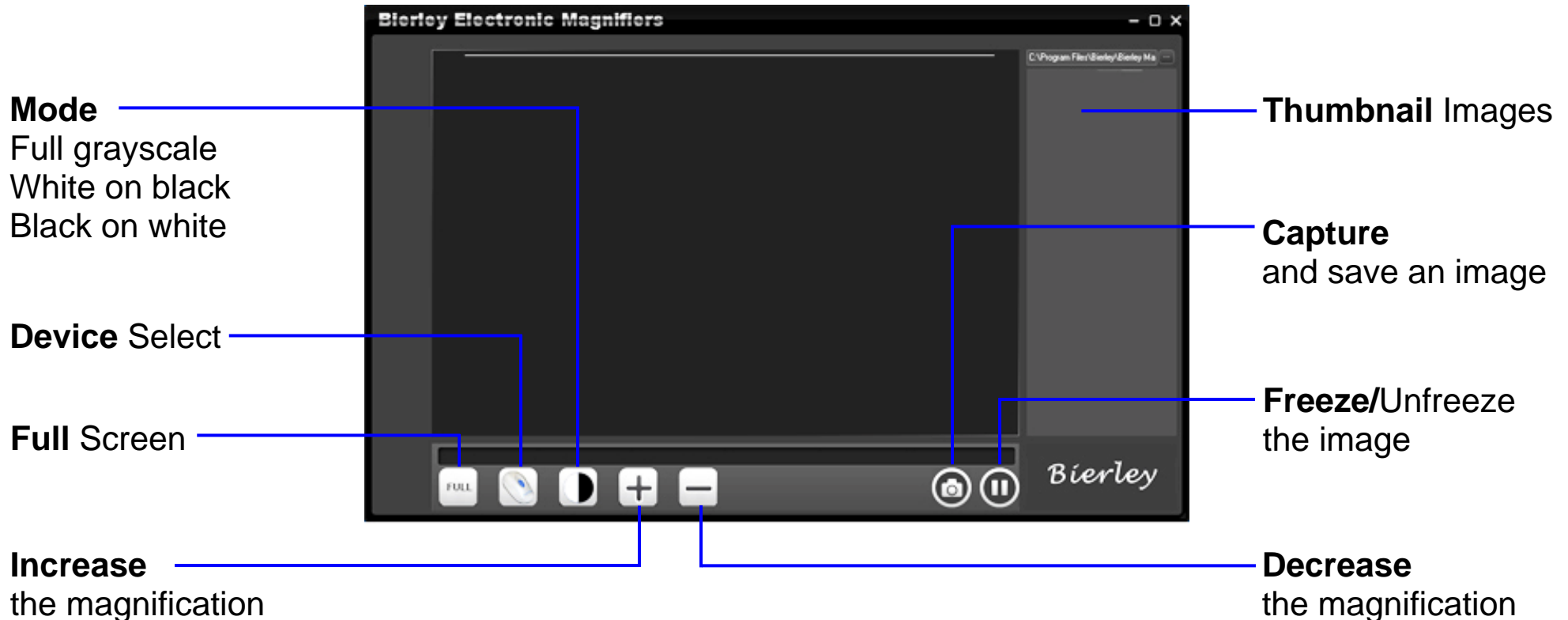

To use: Launch Bierley magnifier software (it will be in your programs in the Bierley directory). Move the magnifier over the material that you wish to read and it will be displayed in the main window.

Note : 1) Each time you launch the software, the magnifier's LED's will flash two times and the screen will flash a few times as the magnifier automatically checks and corrects itself for optimal illumination.
2) Many customers prefer simple products and instructions, if there is anything that you can't understand though, please telephone or e-mail us for a full and clear explanation.

** For desktop computers, only use a rear mounted USB port.