


# CARSON<sup>®</sup>

# MAGNIFREE<sup>™</sup>

2x / 3.5x HANDS-FREE MAGNIFIER


HF-25

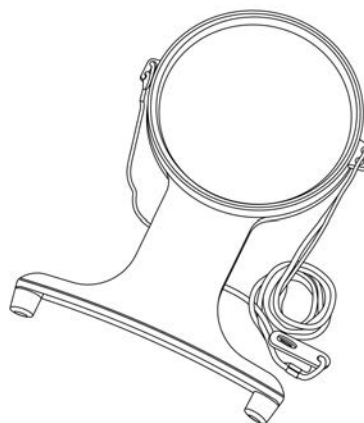

## **Instructions for use:**

### **To Use Hands Free**

Place strap around neck and adjust length as desired. Rest rubber tips on bottom of magnifier against chest (Fig. 1). Rubber feet keep magnifier from slipping.

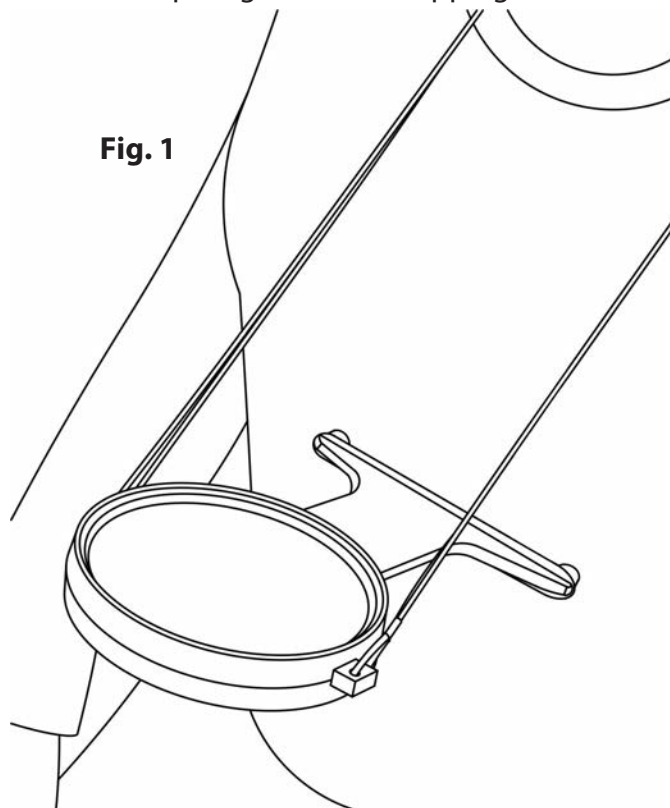

### **Cleaning Your Magnifier**

Clean lens with a soft, clean microfiber cloth by rubbing gently in a circular motion. Moisten the lens cloth with water if needed. Do not use harsh chemicals, solvents or abrasives as they may damage the lens.

### **CAUTION - FIRE HAZARD:**

Keep magnifier away from direct sunlight to prevent accidental fire.

### **Customer Service:**

If you experience any difficulties, please contact us and we will be happy to help you.

#### **In the U.S.A.**

**Phone:** 1-631-963-5000  
**Email:** info@carson.com

#### **In the U.K.**

**Phone:** +44 (0) 1938 557440  
**Email:** uksupport@carson.com

#### **In the E.U.**

**Phone:** +44 (0) 1938 557440  
**Email:** eusupport@carson.com

**For warranty information, visit [www.carson.com/warranty](http://www.carson.com/warranty)**