

SilentCall[™] ALARMDOCK[™]

Smartphone Docking Station with Speaker and Shaker Module

For use with any Smartphone or Tablet*



User's Manual

TABLE OF CONTENTS

Introduction.....	4
Important Safety Instructions.....	4
Danger	4
Warnings	4
Precautions.....	4
USB Charger/Power Cube Specifications.....	5
Smartphone/Tablet Application Requirements.....	5
User Manual	5
AlarmDock Wireless Speaker Specifications	5
AlarmDock Wireless Speaker, Model # PD100-SC	6
AlarmDock Wireless Speaker Application and Setup	6
Bed Vibrator, Model # VIB-PJ.....	7
Shaker Module, # AD200-SM	7
USB Charger/Power Cube, Model # 890010	8
Shaker Module Setup and Pairing.....	8
Pairing Instructions for the Shaker Module	8
FCC Statement.....	9
AlarmDock Wireless Speaker Application Setup Icons	10
Troubleshooting Tips.....	11-12
Warranty.....	back cover

INTRODUCTION

Thank you for choosing the AlarmDock™. This user manual contains useful information on appropriate use and maintenance of this product. In order to benefit from its features, read this manual before first use and save it for further reference.

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING: Read all instructions before using this product.

DANGER:

- Do not let water or any other liquid or flammable detergent enter the power jack to avoid electric shock and/or fire hazard. Do not clean the USB Charger/Power Cube with water or any other liquid or a (flammable) detergent to avoid electric shock and/or a fire hazard.
- To prevent fire or shock hazard do not expose this product to water or moisture. Before connecting to the main power make sure that the requirements specified on the USB Charger/Power Cube correspond to the available power. i.e. AC100-240V~, 50/60 Hz.

WARNINGS

- This product is not intended for use by children or persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the unit by a person responsible for their safety.
- Modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

PRECAUTIONS

Children should be supervised to ensure that they DO NOT play with the AlarmDock or its components.

USB CHARGER/POWER CUBE SPECIFICATIONS

Switching type AC/DC Power Adapter, Rated Input: 100-240 Vac, 50/60 Hz, 0.8A

SMARTPHONE/TABLET APPLICATION REQUIREMENTS

Android™: 4.0 or higher

iPhone®: iOS 8 or later, compatible with iPhone®, iPad®, and iPod touch®.
App size 2.4 MB.

Tablets: Compatible with all makes/models

THIS USER MANUAL DEFINES PROPER CONNECTION AND USE OF:

- AlarmDock wireless speaker, Model # PD100-SC
- Bed Vibrator, Model # VIB-PJ
- Shaker Module, Model # AD200-SM
- USB Charger/Power Cube, Model # 890010
- AlarmDock smartphone application (Android and iOS)

ALARMDOCK WIRELESS SPEAKER SPECIFICATIONS

Speakers: 2 x 40mm

Amplifier power: 2 x 4w

Frequency response: 150Hz - 20kHz

Signal-to-noise ratio: up to 100dB

Distortion: ≤5%

Battery type: polymer Li-ion

Wireless transmitter frequency range: 2.4–2.48GHz

Battery charging voltage: 5V

Battery charge time: 3-4 hours

Battery capacity: 1000 mAh

Wireless transmission distance: 33 feet

Dimensions: 6.2 x 4.2 x 5.6 inches

1 ALARMDOCK WIRELESS SPEAKER, MODEL # PD100-SC, FEATURES:

- 2-in-1, wireless speaker and smartphone stand
- Extra-loud alarm, up to 100 decibels
- Unique, patented design
- Touch-sensitive controls
- 2 broad-frequency response speakers
- Rich, 360°, crystal-clear sound
- Compatible with Android/iOS/Windows®/macOS®
- Built-in, rechargeable Li-ion battery provides up to 10 hours play time
- Wireless and audio (port) connections
- Memory pairing capability. Easy to operate
- Two-way USB port, plays and charges at the same time
- 90 day warranty




ALARMDOCK WIRELESS SPEAKER APPLICATION AND SETUP

You'll need to download the AlarmDock application. For Android™, go to the Google Play™ Store. For iPhone®, go to the iTunes® App Store. Type AlarmDock into the application search field, then look for AlarmDock by Silent Call® Communications. Once you've downloaded the application, you'll need to open it to set up your alarm settings and to pair the wireless Shaker Module. To use the AlarmDock application, one device at a minimum must be paired for it to work. See application requirements under smartphone application requirements above.




2 BED VIBRATOR, MODEL # VIB-PJ, 5 YEAR WARRANTY

Best placement for the Bed Vibrator is under the mattress, between the mattress and box spring. Place in a spot that would be closest to your upper back. Once in place, plug the Bed Vibrator into the end of the Shaker Module input jack. Plug the Shaker Module into the USB Charger/Power Cube, then plug the USB Charger/Power Cube into the 120 volt wall outlet. Very important: "DO NOT USE ANY OTHER MANUFACTURER'S BED VIBRATOR WITH THE SHAKER MODULE, IT COULD DAMAGE THE SHAKER MODULE AND VOID THE WARRANTY".

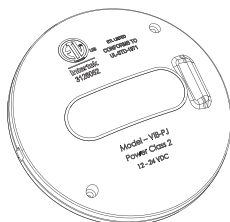


3 WIRELESS SHAKER MODULE, # AD200-SM, 90 DAY WARRANTY

Plug the wireless Shaker Module into the USB Charger/Power Cube. Plug the USB Charger/Power Cube into a 120 volt wall outlet. The USB Charger/Power Cube will keep the Shaker Module powered and ready to activate the Bed Vibrator. When the Shaker Module is in range, approximately 30' from your smartphone, it will automatically pair itself to the smartphone. A blue light on the Shaker Module will light up and flash on & off when ready to be paired. When paired the blue light will stay steady, no flashing.

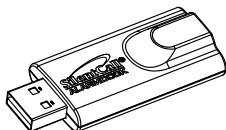

4 USB CHARGER/POWER CUBE, MODEL # 890010, 1 YEAR WARRANTY

The USB Charger/Power Cube is designed to power and charge up to four devices at one time. Charge devices like the AlarmDock wireless speaker, smartphones, and tablets. It also powers the wireless Shaker Module.

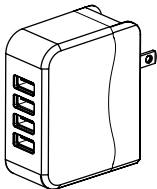

SHAKER MODULE SETUP AND PAIRING

Plug the Bed Vibrator jack into the jack input at the end of the Shaker Module. Next, plug the Shaker Module into the USB Charger/Power Cube and plug the USB Charger/Power Cube into any 120 volt outlet. Make sure the jack on the Bed Vibrator is pushed all the way in to the Shaker Module. Place the Bed Vibrator between the mattress and box spring of your bed. For best results, place the Bed Vibrator under the area where your upper back would be positioned when in bed.

PAIRING INSTRUCTIONS FOR THE SHAKER MODULE

Shaker Module pairing: Once your smartphone is in range of the Shaker Module (approximately 30' or less), the Shaker Module will automatically pair with your smartphone. A blue flashing indicator light will flash on the Shaker Module stick and will change to steady lit, no flashing, when paired. If the blue LED is still flashing, you'll need to turn on the Bluetooth setting in your smartphone's settings.

WARRANTY

The Silent Call AlarmDock wireless speaker is warranted for (see individual items for length of warranty) from the day of purchase to be free of defects in material and workmanship. This warranty is limited to free repair or replacement when the product is shipped prepaid to:

Silent Call Communications Corporation
5095 Williams Lake Road
Waterford, Michigan 48329

This warranty is void when damage is caused by misuse, neglect, fire, excessive heat, water, or by improper installation or operation. Silent Call will not be responsible for the cost of parts or labor and/or service charges which are performed by others.

To obtain warranty service, call toll free 1-800-572-5227 or email us at technicalsupport@silentcall.com

After this warranty expires, Silent Call will repair or replace any Silent Call product for a fixed cost.

www.silentcall.com

Silent Call® Communications Corporation
5095 Williams Lake Road
Waterford, Michigan 48329

The logo for Silent Call Communications features the company name in a bold, sans-serif font. Above the text is a stylized, curved graphic element that resembles a signal or a wave. The word "SilentCall" is on the top line, and "Communications" is on the bottom line, both in black.

SilentCall
Communications