



The catalog of products for the visually impaired and hard of hearing

Customer Service: 800-468-4789

Facsimile: 877-498-1482

Wednesday, October 15, 2014

A2005 Spectacles Flip Up-Clamp for Monoculars

Mounting monocular to adapter:

- 1. Turn the monocular so it is fully extended.**
- 2. Give the monocular a firm turn to loosen the eyepiece and unscrew the eyepiece from the monocular**
- 3. Screw the monocular into the adapter**

Applying the clamp onto glasses:

- 1. Flip up silver circular bar in the back of the unit**
- 2. With finger on the circular bar and thumb on front silver bar squeeze together.**
- 3. Apply to glasses**

By loosening the side screws the user may adjust the adapter. This is important so the telescope will sit in front of the pupil.