



A better way to stay in touch™

www.silentcall.com

Sidekick II™

Receiver

Model # SK09214-2



318MHz

Sidekick II Receiver

Model # SK09214-2

(318 MHz Receiver)

Setup and Operation:

1. Plug the power supply pin jack into the power supply terminal located next to the Power ON/OFF switch on the back of the receiver. (see page 4 for diagram) Plug power supply into a wall electrical outlet.
2. Turn the Power ON at the ON/OFF SWITCH located at the bottom left corner on the back of the receiver. The front panel display will show the time 11:59AM.
3. Now set the clock time by pressing and releasing the MODE button once. The Orange LED will light up to let you know that you are in "Set Clock Time" mode.
4. Press and release the minute button to set the minutes. Press and release the hour button to set the hour.
5. When you are finished setting the minutes and hours, press and release the MODE button three times to exit the Setup Mode.

NOTE: Any time you unplug your Sidekick II Receiver from the wall outlet or turn the power off, the next time you turn your Sidekick II Receiver on again, you will need to reset the clock. If you are using the Battery Backup feature, you will not need to do this.

6. All of your Silent Call transmitters come pre-programmed from the factory to operate your Sidekick II Receiver. If you need to set your transmitters to a different address, please refer to the section in this manual on programming transmitters. The only reason you would need to do this is if someone in your area is using Silent Call products and they are interfering with your products.

Battery Back-up Installation Instructions

If you purchased the battery backup pack at the same time as your Sidekick II Receiver, it was pre-installed for you at the factory. If you purchased the battery back-up, Model # BU-10 at a later time, follow the installation directions.

To gain access to the battery compartment remove the screw at the top of the cover on the back of the receiver. Discard this back and use the new one that came with the backup battery. Plug the female connector from the battery pack onto the male connector pin in the battery compartment. Be sure to line up the pins with the holes in the connector. It will only fit one way. Replace the New battery compartment door and secure the screw. (See page 4 for screw and pin locations)

You will notice an On/Off switch on the right hand bottom corner of the battery compartment door. Slide the switch to the right which is the ON position. It is important that the battery back-up feature be turned "ON" for proper operation.


When a power outage occurs, the strobe light on the Sidekick II Receiver will turn an Amber color indicating that you receiver is operating on battery backup in Stand By Mode. The Ni-Cad battery, Model # BU-10, has been tested and listed by ETL to keep the Sidekick II receiver operational for 24 hours in Stand By mode and 15 minutes in full operational mode, (both the strobe and vibrator are activated).

The battery back-up is automatically being charged when the Sidekick II power supply is plugged into a standard electrical wall outlet.

The BATTERY ICON will display itself on the LCD screen once the battery back-up level has reached 2/3 of its discharge. However, the BATTERY ICON will not be displayed until this condition is reached. When the Amber LED is on but the BATTERY ICON is not displayed on the screen, this means that the battery is at a full or close to full charge.

Setting Alarm Clock:

1. Press and release the MODE button twice.
2. The Green LED will light up indicating that you are in the "SET UP" mode. The display will show the present Alarm Clock Time setting.
3. Press and release the MINUTES button to set minutes and press and release the HOURS button to set the hours.
4. Once the Alarm Clock Time has been set, press the MODE button 2 times to exit the "SET UP" mode.
5. To turn the ALARM on or off, press and release the "Alarm Clock" button. When the alarm is turned ON the BELL ICON will appear on the screen.

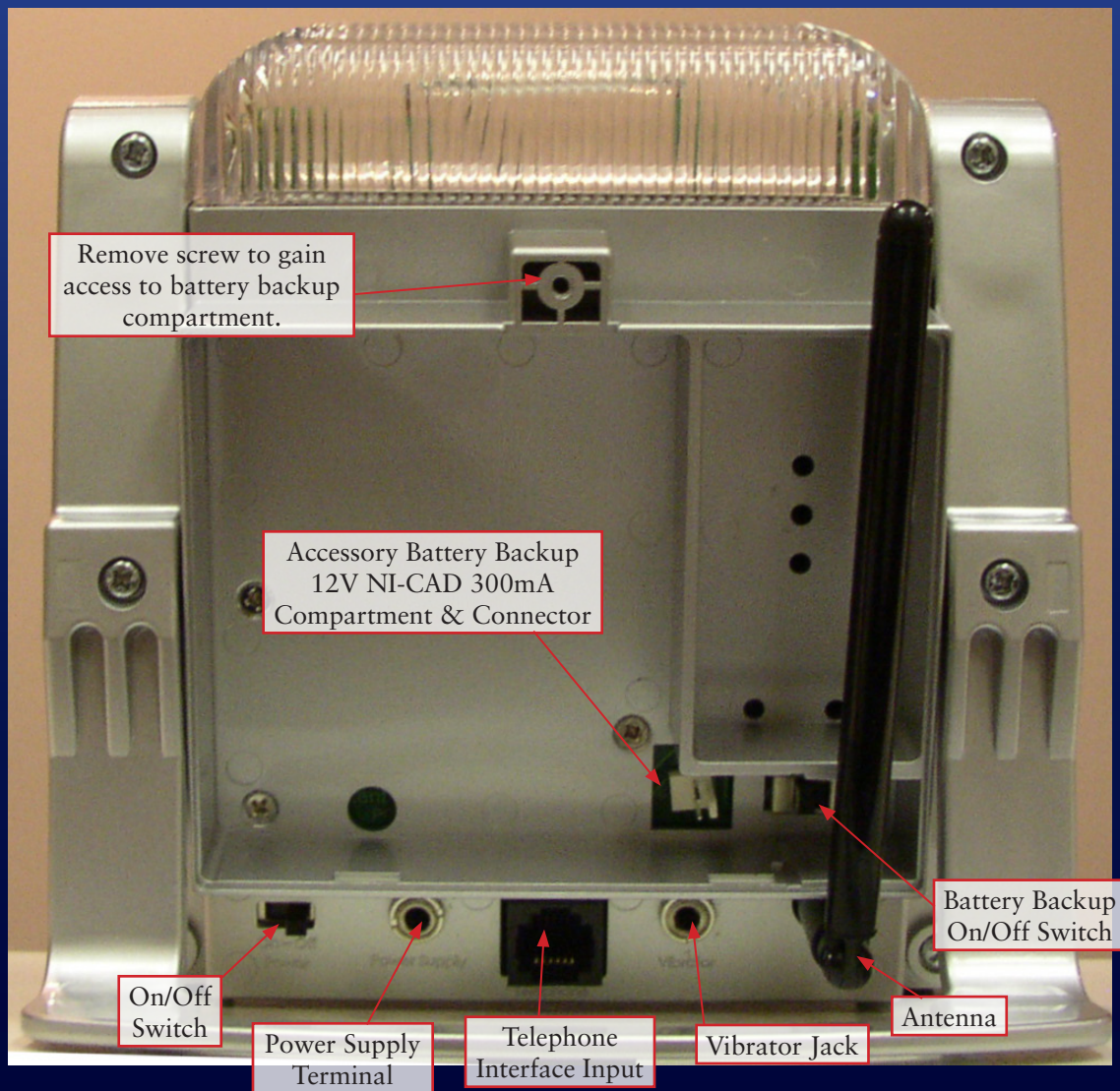

- When your alarm is activated the strobe light will flash, the bell icon will display and the bed vibrator (optional) will shake.

LCD Display:

The LCD Display screen is back lighted. The back light brightness is automatically controlled by the light level in the room. When the room becomes dark, the back lighting comes on. When the room becomes light, the back lighting goes off.

Telephone Direct Interface:

The Sidekick II Receiver has the ability to plug a standard telephone directly into the receiver and it will notify you each time the telephone rings. (see Diagram Below)

Plug the supplied dual connector into your phone wall outlet. Then plug the wire from your telephone into one side of the dual connector. Next, take the telephone wire that came with your Sidekick II Receiver and plug it into the other side of the dual connector. (See illustration #1) Finally, plug the other end of that cable into the back of the Sidekick II Receiver (See illustration #2)

The receiver will notify you in the same way as if you were using a Telephone Transmitter (See section on Notifications, pg 6).



Illustration #1

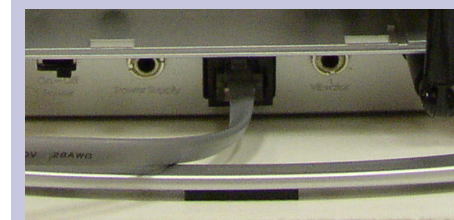











Illustration #2

Transmitters

When the Sidekick II Receiver detects an incoming signal from any of the Silent Call 318 MHz transmitters it will notify you by:

| Transmitter Type | Icon | Notification |
|--------------------------------|---|---|
| Alarm Clock |  | Bell Icon, Strobe flashes <i>(Vibrator Optional)</i> |
| Sound Monitor Transmitter |  | Sound Icon, Orange LED, Strobe flashes <i>(Vibrator Optional)</i> |
| Telephone Transmitter |  | Phone Icon, Green LED, Strobe flashes <i>(Vibrator Optional)</i> |
| Doorbell Transmitter |  | Door Icon, Yellow LED, Strobe flashes <i>(Vibrator Optional)</i> |
| Door/Window Access Transmitter |  | Door Icon, Yellow LED, Strobe flashes <i>(Vibrator Optional)</i> |
| TransMATTer Transmitter |  | Door Icon, Yellow LED, Strobe flashes <i>(Vibrator Optional)</i> |
| Weather Alert Transmitter |  | Weather Icon, 2nd Red LED, Strobe flashes <i>(Vibrator Optional)</i> |
| Carbon Monoxide Transmitter |  | CO Icon, 2nd Red LED, Strobe flashes <i>(Vibrator Optional)</i> |
| Smoke or Fire Transmitter |  | Fire Icon, 4th, 5th, or 6th Red LED, Strobe flashes <i>(Vibrator Optional)*</i> |

*If you are using the optional Bed Vibrator and your Sidekick II Receiver detects an incoming signal from any of the Silent Call 318 MHz transmitters, along with the above Notifications, the bed vibrator will also pulse in a specific way for each transmitter.

| Transmitter Type | Pulse Pattern | Short Pulse • Long Pulse — |
|--------------------------------|---------------------------------------|-------------------------------|
| Alarm Clock | Two Short Pulses and One Long Pulse | • • — |
| Sound Monitor Transmitter | Three Short Pulses | • • • |
| Telephone Transmitter | One long pulse | — |
| Doorbell Transmitter | Two Short Pulses | • • |
| Door/Window Access Transmitter | Two Short Pulses | • • |
| TransMATTer Transmitter | Two Short Pulses | • • |
| Weather Alert Transmitter | One Short Pulse and One Long Pulse | • — |
| Carbon Monoxide Transmitter | Three Short Pulses and One Long Pulse | • • • — |
| Smoke or Fire Transmitter | Continuous Short Pulses | • • • • • • • • |

Transmission Range:

Transmission range refers to the distance a Silent Call Transmitter can communicate with the Sidekick II Receiver. Range is measured in an outdoor environment without obstructions from metal objects or buildings. The range for the Sidekick II Receiver is up to 500 feet. The actual range inside a building will vary depending on many factors outside of one’s control.

Cancelling An Incoming Alert

To cancel an alert, press and release the mode button once.

Programming the transmitters:

All Silent Call 318 MHz Transmitters come pre-programmed at the factory. You do NOT need this step unless you need to change the address due to someone in your area that has Silent Call products and they are interfering with your equipment.

1. Make sure that all Silent Call transmitters in the area are turned off.
2. Perform the ALL ERASE function (see pg 8)
3. Select the transmitter you wish to program with the Sidekick II Receiver.
4. Locate the switch on the transmitter circuit board that has 5 small dip switches. Set the switches to any combination that you want. For Example: 1,2 On 3,4,5 OFF. (Do not set the switches to all ON or all OFF as this can cause interference). This gives your transmitter an “address”.
5. When programming the transmitter, make sure that the transmitter is near the Sidekick II Receiver.
6. On the Sidekick II Receiver, press and release the MODE button until the Yellow LED lights up and “PPP” is displayed on the screen. This means you are now in program mode.
7. On the transmitter, press the test button so that it sends a signal to the Sidekick II Receiver. The unit will flash its strobe, turn on the appropriate LED and display Icon for the transmitter that you used. Your transmitter is now programmed to work with your receiver. With all of your transmitters set to the same address it is not necessary to program every transmitter to the receiver.

All Erase:

Provisions are made to erase all programmed transmitter addresses. The ALL ERASE function is performed as follows:

1. Turn off the Sidekick II Receiver by turning the AC power switch to the off position. Turn back-up battery to off position (if you have the backup battery installed). An indication of power being off would be that the amber light on the strobe lens is off.
2. Press and hold the MINUTES button. DO NOT RELEASE THE BUTTON
3. Turn the power back on while holding the MINUTES button in.
4. Once power has been on for 5 seconds, release the MINUTES.

The all erase function is complete and the unit will now need to be re-programmed with its transmitters.

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

Limited Warranty

Your Silent Call product is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect. We honor the smoke detector manufacturer's warranty for their smoke detector. Only Silent Call products are warranted for 5 years.



5095 Williams Lake Road
Waterford, MI 48329

Phone 800-572-5227 v/tty
www.silentcall.com