Sound Monitor

Model# SM5-SS


Please read all of the instructions thoroughly before using your Sound Monitor.

Installation

- 1. Remove your Sound Monitor from the packaging.
- 2. Place the Sound Monitor as close to the sound source as possible that you want to monitor (but not within a baby's reach). For example, place it on a dresser next to a baby's bed or next to a telephone, alarm clock or apartment doorbell speaker.
- 3. The Sound Monitor can be mounted to any flat surface with the supplied Velcro strips or you can hang it on a wall using the access hole on the back of the case. This step is not necessary if you will be moving the Sound Monitor from location to location.
- 4. Avoid placing the Sound Monitor on a metallic surface. Metal will interfere with the signal transmission and shorten the signal range.

Operation

- 1. The Sound Monitor operates on 2 AA Alkaline batteries (included).
- 2. There is a Yellow LED (low battery indicator light) on the face of the transmitter. When this light comes on you know that the batteries need replacing.
- 3. Slide the on/off switch on the side of the Sound Monitor to the ON position.
- 4. Any continuous loud sound will cause the Sound Monitor to activate and the Red LED on the face of the transmitter will come on. The transmitter then sends a wireless signal, to any Signature Series Receiver activating the receiver.
- 5. Transmission range is determined by which Signature Series receiver you are using.
- 6. The Sound Monitor is designed so that brief sounds such as door slams, regardless of how loud they are, will not activate the Sound Monitor.
- 7. The Sound Monitor leaves the factory set to 50% sensitivity, which should be adequate for almost all situations. If for some reason the sensitivity needs to be changed, there is an adjustment dial located behind the battery door. Rotating the adjustment dial clockwise increases the sensitivity.
- 8. A new feature of the Sound Monitor is the ability to set the Sound Monitor to detect a specific sound. Remove the battery cover and at the bottom left hand corner there is a 3 position switch. Switch # 1 is for Baby Cry, Switch #2 is for Telephone and Switch # 3 is for Doorbell.

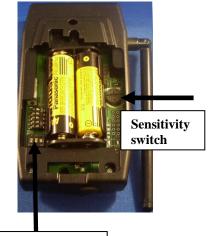

3 position switch

Address Switch Settings

The Silent Call system is digitally encoded. All Silent Call receivers and transmitters are tested and leave the factory programmed to a factory default address. *You do not need to change the address unless someone in your area has Silent Call products and they are interfering with your equipment.*

- 1. Make sure that all Silent Call transmitters in the area are turned off.
- 2. Located on the back of the transmitter case is a removable access panel. Remove the access panel and take out the batteries. Note that you MUST remove the batteries first or the switch setting will not take effect.
- 3. Locate the address switch on the transmitter circuit board that has 5 small dip switches. Set the switches to any combination that you want. For Example: 1, 2 ON 3, 4, 5 OFF. This gives your transmitter an "address". Note: Do not set the switches to the all "ON" or all "OFF" position.
- 4. Reinstall the batteries and replace the access panel.
- 5. Refer to your specific Signature Series Receiver instruction manual for programming your receiver to your newly changed transmitter address.



5095 Williams Lake Road, Waterford, MI 48329 (800) 572-5227 v/tty (248) 673-7360 fax

Website: www.silentcall.com Email: silentcall@silentcall.com

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY), by Email at support@silentcall.com or visit us on the web at www.silentcall.com.

Limited Warranty

Your Sound Monitor is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.



5095 Williams Lake Road, Waterford, MI 48329 (800) 572-5227 v/tty (248) 673-7360 fax